

# SuperNova<sup>®</sup> Case Dry<sup>™</sup>

## Ultra Concentrated Drying and Sheeting Agent

### SuperNova<sup>®</sup> Case Dry<sup>™</sup> Drying Agent Specifications

#### Introduction:

SuperNova<sup>®</sup> Case Dry<sup>™</sup> ultra concentrated drying and sheeting agent is designed to bead water and enhance natural or forced air drying. It is carefully formulated to produce the lowest surface tension to optimize drying efficiency and prevent water spots.

#### Concentration:

2 fl. oz/ 100 gallons (16 ml/ 100 liters)

#### Instructions for Use:

- Dilute product according to directions
- Mix with water that is 130° F to 190° F (54° C to 88° C)
- For automated cart and utensil washers, set timing rate to approximately
- 20 to 25 seconds

#### Storage Conditions:

- Store at room temperature (10°C to 25°C) away from heat source
- Keep container closed when not in use

#### Compatibility and Safety:

- Formulated with U.S. EPA Safer Choice listed chemicals for a safer product
- Environmentally friendly
- Non-corrosive, non-staining, non-sticky
- Compatible with all metals including anodized aluminum and stainless steel
- Compatible with all surface materials including plastic and glass
- Tested successfully for compatibility over a one year period

#### Caution:

- Thoroughly review product Safety Data Sheet
- Wear appropriate personal protective equipment (PPE) when handling contaminated devices and chemicals
- If eye contact or irritation occurs, rinse well



**CSNCD1W**

**Technical Breakdown:**

- Use in the final rinse in cart washers and utensil washers
- Water dispersible
- Water-beading action minimizes water spotting
- Fast acting, excellent sheeting action accelerates drying time
- pH neutral, non-corrosive to surgical instrumentation or equipment
- Super concentrated formulation for staff safety and ergonomic preference
- Super concentrated formula provides more uses per container



Designed for health care.  
Use in cart and utensil washers  
for the final rinse.

For more information, please see SuperNova® Case Dry™ Drying Agent Safety Data Sheet

<i>CSNCD1GW</i>	SuperNova® Case Dry™ Drying and Sheeting Agent 4x1 gallon per case
<i>CSNCD1W</i>	SuperNova® Case Dry™ Drying and Sheeting Agent 2x2.5 gallons per case

Case Medical Inc.  
50 West Street  
Bloomfield, NJ 07003  
www.casemed.com

U.S. and Canada  
1-888-227- CASE  
Worldwide  
+1 201-333-1999

Copyright © 2017- 2023 Case Medical  
Inc. All rights reserved. Rev C 11/23