

Case Solutions[®] SuperNova[®]

InstruCreme[®] Lubricant



SuperNova[®] InstruCreme Lubricant Specifications

Introduction:

Ready-to-use SuperNova[®] InstruCreme[®] Lubricant is a specially formulated aqueous emulsion designed to ease stiffness, prevent rust, and extend the life of surgical instruments. InstruCreme lubricant maintains a smooth working action, is pH neutral and water dispersible, has a natural odor, and is milky in color.

SuperNova InstruCreme Lubricant is environmentally friendly, formulated with safer ingredients, and has been awarded the U.S. EPA SAFER CHOICE label. Our lubricant keeps working elements of surgical devices smooth acting to maintain a long useful life. It is a non-corrosive, non-staining, nonsticky, pH neutral aqueous emulsion that serves as a rust inhibitor.

SuperNova InstruCreme Lubricant is compatible with stainless steel, aluminum, tungsten carbide, copper, and brass.

Directions for Use:

- Keep on hand at each prep and pack station to apply as needed to box locks and other moving parts and latches
- Apply manually to the working parts of instrumentation
- Use lubricant after thorough cleaning and rinsing
- Allow to dry then proceed to autoclave

Storage Conditions:

- Store at room temperature (10^oC to 25^oC) away from heat source
- Keep container closed when not in use

Compatibility and Safety:

- Non corrosive
- Lower toxicity
- Compatible with stainless steel, aluminum, tungsten carbide, copper, and brass



CSL014

U.S. EPA
Safer Choice
Awarded



Caution:

- Ingredients may cause severe skin burns and eye damage
- Harmful if swallowed
- Hazard class not classified



Technical Breakdown:

- Neutral pH 7.0 to 8.0
- Exceptional performance in all water types and at varying temperatures
- Natural odor, milky color
- Environmentally friendly with safer chemicals
- Water dispersible

For more information, please see SuperNova® InstruCreme Lubricant Safety Data Sheet

<i>CSL02</i>	SuperNova® InstruCreme® Lubricant 12x2 oz bottles per case
<i>CSL014</i>	SuperNova® InstruCreme® Lubricant 12x34 oz bottles per case

*Includes 12 Sprayers

Case Medical Inc.
50 West Street
Bloomfield, NJ 07003
www.casemed.com

U.S. and Canada 1-888-227- CASE
Worldwide +1 201-333-1999

Case Solutions, InstruCreme, and SuperNova
are registered trademarks of Case Medical.
Copyright © 2025 Case Medical Inc.
All rights reserved. Rev D 6/25