

# Case Solutions® *SchmutzOff*®

## Descaler and Stain Remover

### SchmutzOff® Specifications

#### Introduction:

SchmutzOff® is a combination descaler, cleaner, and stain remover that restores finish to automated washers, stainless steel instruments, counters, carts, and autoclave walls. Specifically formulated to remove scale, stains, organic matter, and mineral deposits from stainless steel surfaces, SchmutzOff descaler also refreshes autoclave by removing noxious odors from walls. By reconditioning stainless steel devices, SchmutzOff descaler restores the original luster and device functionality.



CSSTR011W

#### Instructions for Use:

- Use as descaler, stainless steel cleaner, and instrument reconditioner
- For reconditioning of stainless steel instruments, soak or add to sonic for a minimum of 10 minutes
- For soaking and manual cleaning, dilute product 1:6
- For highly stained surfaces, apply directly
- Rub with soft cloth or non-metallic brush
- For removal of scale inside washer, either apply directly or connect to reconditioning wash cycle
- Rinse well after use
- Follow with detergent wash to remove any residue, then thoroughly rinse
- For instrumentation, follow with normal cleaning procedures per manufacturer's IFU

#### Storage Conditions:

- Use PPE when applying SchmutzOff® descaler
- Store at room temperature (10°C-25°C) in a well ventilated space away from heat source
- Keep container closed when not in use
- Avoid contact with rubber gaskets

#### Storage Conditions:

- Compatible with stainless steel
- Acid based cleaner—do not use on anodized aluminum products or silicone materials
- Tested successfully for material compatibility over a one year period

## Caution:

- Wear appropriate personal protective equipment (PPE) when handling contaminated devices and chemicals
- Wear protective clothing, gloves, eye, and face protection
- Avoid contact with skin
- Avoid splashing in eyes
- Do not breathe mist, spray, or vapors
- May be corrosive to metals
- Do not use on aluminum, chrome, or silver plate

## Technical Breakdown:

- Acid based liquid descaler
- Contains surfactant blend, phosphoric acid, and high purity water
- pH 2-2.5

For more information, please see SchmutzOff® Descaler Safety Data Sheets

<i>CSSTR24</i>	SchmutzOff® Descaler & Stain Remover 12x24oz per case*
<i>CSSTR011W</i>	SchmutzOff® Descaler & Stain Remover 4x1 gallon per case
<i>CSSTR016W</i>	SchmutzOff® Descaler & Stain Remover 2x2.5 gallon per case

\*Includes 12 sprayers

Case Medical Inc.  
50 West Street  
Bloomfield, NJ 07003  
www.casemed.com

U.S. and Canada  
1-888-227- CASE  
Worldwide  
+1 201-333-1999

CASE Solutions is a registered  
trademark of Case Medical.  
Copyright © 2017- 2023 Case Medical  
Inc. All rights reserved. Rev D 11/23