

Case Solutions® *Penta Wipes*

Disposable Surface Cleaning Wipes

Highly Saturated with PentaZyme® Multi-Enzymatic Cleaner

CASE Solutions® Penta Wipes Cleaning Wipes Specifications

Introduction:

CASE Solutions® Penta Wipes are premoistened, non-woven, disposable surface cleaning wipes. Each single-use wipe is impregnated with U.S. EPA Safer Choice awarded PentaZyme® solution, a biodegradable, freerinsing, multi-enzymatic cleaning solution with a two minute contact time. Penta Wipes are compatible with all medical devices and designed to be used for cleaning rigid and flexible endoscopes, and environmental surfaces. Penta Wipes are compatible with the following materials: metals (such as anodized aluminum and stainless steel), plastic, silicone, glass, and rubber.

Instructions for Use:

- **Surface Cleaning**
 - Remove gross soil
 - Wipe all surfaces with Penta Wipes disposable wipe
 - Additional wipes may be necessary for further cleaning
 - Allow moisture to remain on surface for a minimum of two (2) minutes
 - Rinse well
 - Dry thoroughly
- **Endoscope Reprocessing**
 - Wipe surface of endoscope with Penta Wipes disposable wipe, moving from the clean end of the endoscope to the soiled end
 - May be used as a pretreatment step to remove soil from devices at point of use in patient or procedure rooms
- **Sterilization Containers and Trays**
 - Rinse off gross soil
 - Disassemble all components
 - Wipe all surfaces thoroughly with Penta Wipes disposable wipe
 - Rinse and dry
 - Follow with Case Solutions® Alcohol Wipes to facilitate drying



CSWE002

Utilizing U.S. EPA Safer Choice awarded PentaZyme® Multi-Enzymatic Cleaner at use dilution



Do Not Flush

Note: Penta Wipes may be used to cover soiled items during transport from the operating room or procedure room to the decontamination area. They will help keep instruments moist and the enzymes and surfactants facilitate the cleaning process

Directions:

- Open packaging and remove lid
- Peel back tamper proof liner
- Pull up corner of center wipe and thread through slits in top of canister
- Pull wipe through at a slight angle
- To close, snap cover tab securely

Storage Conditions:

- Store at room temperature (10^oC to 25^oC) away from heat source
- Keep container closed when not in use
- Do not reuse empty container

Compatibility and Safety:

- Non-corrosive, lower toxicity
- Compatible with all metals, silicone, glass, and all plastic materials
- Tested successfully for compatibility over a two year period
- PentaZyme[®] Cleaner is U.S. EPA Safer Choice awarded for sustainability and safer chemistry

Caution:

- Wear personal protective equipment (PPE) when handling contaminated medical devices and chemicals or cleaning contaminated surfaces
- Do not push finger through slot on lid
- Avoid contact with eyes
- Use gloves to avoid sensitization
- Not to be used on skin
- Do not reuse wipes are provided single use
- Dispose of used wipes in accordance with local regulations

For more information, please see Case Solutions[®] Penta Wipes Safety Data Sheets

<i>CSWE002</i>	Case Solutions [®] Penta Wipes, 6 canisters (80 wipes each) per case
----------------	---

Case Medical has been recognized by the U.S. EPA, as a 2023 Safer Choice Partner of the Year, for developing environmentally oriented medical instrument cleaners to safeguard human health. PentaZyme[®] has fully achieved the objective as an effective, environmentally preferred product.